

To Change Settings

All user input and feedback is provided through the PENDANT.

Extension



Press/Hold Extension Button



Press UP/Down arrows to desired angle

Flexion



Press/Hold Flexion Button



Press UP/Down arrows to desired angle

Carriage Speed



Press/Hold Speed Button



Press Up/Down arrows to desired speed

Extension / Flexion Delay



Allows the user to program the carriage to stop at both the extension and flexion angles for a programmed period of time.

Comfort Zone



Pressing this button will temporarily decrease the flexion ROM.

Start/ Emergency Stop



Pressing this button once will STOP the unit and pressing this button a second time will REVERSE the direction of the carriage. **This button functions as BOTH the START and STOP switch.**



QUICK START MENU

Refer to the OptiFlex users manual for more detailed instruction.

Special Functions

Mode



To enable/disable the Fast Back, Oscillation Zone, or Progressive ROM functions, Press/Hold the Mode button together with the selected option(s).

Fast Back



When enabled, the carriage speed will decrease to half the programs speed for the last 15° of flexion.

Oscillation Zone



When enabled, the carriage will oscillate three (3) times in the last 10° of flexion then continue with the full ROM. This cycle is repeated during the entire treatment.

Progressive ROM



When enabled, the unit will automatically increase the programmed flexion angle by 1° every hour up to a maximum of 5° per day.

Reset

Pinhole

Under the display. Clears previous treatment settings and completed cycles by pressing it with a paperclip.

Refer to the OptiFlex user manual for more detailed instructions or call Chattanooga Group at **1-800-592-7329**.



4717 Adams Road

P.O. Box 489

Hixson, TN 37343 U.S.A.

www.chattgroup.com

1-423-870-2281

1-800-592-7329 U.S.A.

1-800-242-8329 FAX

1-800-361-6661 CANADA